

Comfort Fit BLU 2 Nitrile Examination Gloves



Order No.	Size
520-45	X-SMALL
521-45	SMALL
522-45	MEDIUM
523-45	LARGE
524-45	X-LARGE
525-45	2X-LARGE

- 100% Nitrile contains no plasticizers, silicone, natural latex rubber, powder, or protein, reducing Type I latex and Type IV chemical allergic reactions and contact dermatitis.
- Effective barrier from blood borne pathogens.
- Textured fingertips for increased grip sensitivity-wet or dry.
- Durable with tactile sensitivity and soft feel.
- High tensile strength creates less chance of tearing.
- Beaded cuff allows for easy donning of gloves.
- Extra soft, stretchy glove reduces hand fatigue.
- 9 1/2" length, ambidextrous gloves fit either hand.
- Blue color.
- Layer packed for easy dispensing, reducing wrinkles and clumping.
- 200 gloves per box, 10 boxes per case, 2,000 gloves per case, sizes XS, S, M, L, X-L.
- 180 gloves per box, 10 boxes per case, 1,800 gloves per case, size 2X-L.
- Complies with FFDCA for food contact.
- Meets ASTM D6319.
- Meets ASTM D6978 for Fentanyl Exposure.
- Meets ASTM D6978-05 for Chemotherapy Drugs. List available upon request

Comfort Fit Nitrile Powder Free Examination Gloves

Chemotherapy Drug Permeation Results

CHEMOTHERAPY DRUG	MINIMUM BREAKTHROUGH DETECTION TIME (Minutes)
Carmustine (BCNU), 3.3 mg/ml *	22.8
Thiotepa, 10.0 mg/ml *	54.6
Cisplatin, 1.0 mg/ml	No breakthrough up to 240 minutes
Cyclophosphamide (Cytoxan), 20.0 mg/ml	No breakthrough up to 240 minutes
Dacarbazine (DTIC), 10.0 mg/ml	No breakthrough up to 240 minutes
Doxorubicin Hydrochloride, 2.0 mg/ml	No breakthrough up to 240 minutes
Etoposide (Toposar), 20.0 mg/ml	No breakthrough up to 240 minutes
Fluorouracil, 50.0 mg/ml	No breakthrough up to 240 minutes
Paclitaxel (Taxol), 6.0 mg/ml	No breakthrough up to 240 minutes

Caution: Gloves used for protection against Chemotherapy drug exposure must be selected specifically for the type of chemicals used. Review material safety data sheets for the drugs being used to determine the required level of protection. *Warning: Do not use with Carmustine (BCNU) and Thiotepa.